

ILLUMINATED YANKAUER SUCKER



MFG. LIC. NO: MFG/MD/2019/000090

Part Number: CM101

LOT BATCH NO. :

 MFG. DATE :

 EXP. DATE :

Mfr by:  **ANGIPLAST**
Private Limited
An ISO 13485 Certified Company

Plot 4803, Phase IV, G/I.D.C Vatva - Ahmedabad - 382 445 INDIA.
Consumer Care: +91 9427606238 - Fax: +91 - 79 - 35841009
Web: <http://www.angioplast.com>
LABEL REV/YS/1/25-05-15 MADE IN INDIA

Manufactured
exclusively for:

 **constructmedical**

11/118 Church Street - Hawthorn VIC 3122 - Australia
Web: <https://www.constructmedical.com>
Ph: +61 3 9029 0928

ILLUMINATED YANKAUER SUCKER

For airway management use only

- * Dimension: Size 8.0mm
- * Non-toxic/Latex Free
- * Single patient use only - Discard after use
- * Non-sterile for use in airway management only. Not for use in sterile surgical field.

⚠ CAUTION

- * The device must only be used by trained, qualified staff
- * Product contains alkaline batteries which may be removed prior to incineration. Please discuss with your waste management provider.

⚠ WARNING

- * Do not reuse. Reuse of the device may cause injury, illness or even death because of biological cross-contamination.
- * Use only for the intended purpose. Use of this device for any other purpose could result in harm to the user or the patient.

INSTRUCTIONS FOR USE

For airway management use only

- * Open the Yankauer suction catheter packaging to expose the connection port only.
- * Ensure the catheter is connected to the suction tubing, and the suction tubing is connected to an appropriate suction device.
- * Test for adequate suction by occluding the catheter's side port.
- * Remove the suction catheter from the packaging.
- * With the side port remaining open, gently insert the catheter's tip into the patients oral cavity.
- * Activate suctioning whilst gently withdrawing the catheter from the oropharynx
- * The area of use may be illuminated by pressing the button situated on the top of the suction handle.
- * If there is a requirement for removal of batteries prior to disposal, the cover may be lifted from the handle using the tab at the rear of the cover. The battery pack may then be broken out of the handle.

PEEL OPEN